

Advanced Radial Shockwave Therapy



enPuls Pro

Zimmer
MedizinSystems

enPuls*Pro*
the next generation

Radial Shockwave Therapy

An Effective Alternative
For Treating Chronic
Soft Tissue Pain



Principle of Radial Shockwave Therapy

The hand piece contains a projectile that is accelerated through the electromagnetic transfer of kinetic energy. This kinetic energy is then transformed into impact energy in the applicator head. The impact energy delivered from the applicator head results in radial pulses in the target tissue. Radial Shockwave Therapy provides an excellent alternative to treating muscle tissue. The energy treats muscle pain and helps to increase circulation in the target tissue.

Mechanical energy

The enPuls*Pro* device delivers energy into the target tissue where traditional soft tissue therapy techniques cannot reach. This is all done with little or no stress on the therapist's hands. A heavy duty, ergonomic aluminum hand piece delivers optimal energy to the patient while providing optimal protection against vibrations to the therapist's hands.

Time savings and efficiency with better outcomes

Adding the enPuls*Pro* Radial Shockwave Therapy unit to your facility will decrease the treatment time for chronic soft tissue pain patients and increase your efficiency. This results in better outcomes for you and for your patients.

Zimmer
MedizinSystems

Stimulating and gentle on the patient...

Exceptionally Powerful Technology

enPulsPro Softshot technology

enPulsPro creates unique radial pulses by using a projectile 5 times heavier than standard air compressor systems and a reduced final velocity upon contact with the applicator. Leveraging the force of the accelerator and the mass of the projectile, the enPulsPro system ensures efficient energy transfer from the hand piece to deep tissue structures, providing a comfortable, effective treatment experience for patients that surpasses comparable systems.

Increased patient comfort

Less pain, same energy – that's enPulsPro Softshot technology. Energy is maintained by a longer pulse rise time. This eliminates the need for unpleasant high peak pressure amplitudes. The lower peak and longer rise time minimize the stinging and painful sensation that is normally associated with this type of therapy. Increased patient comfort results in easier treatments at higher energy levels, leading to successful therapy.



F=ma: Softshot technology

Zimmer MedizinSystems electromagnetic technology uses a projectile that has a 5 times greater **mass** than pneumatic systems. This makes for equal **Force** using 5 times less acceleration.

Less acceleration means a softer, more comfortable pulse on the surface of the skin. We call this **Softshot technology**.



Pneumatic projectile (left) vs. enPulsPro's electromagnetic projectile (right) - 5 times heavier

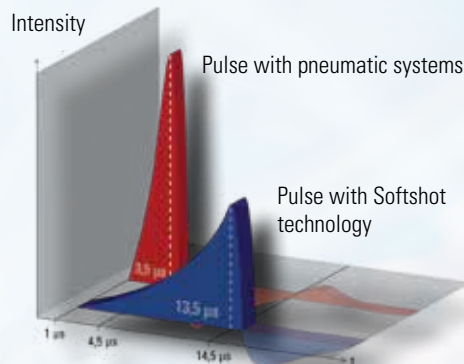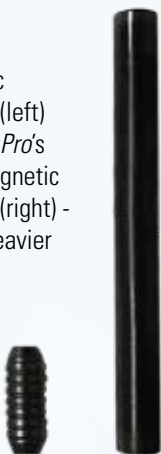

Enhanced patient comfort due to longer rise time (pulse width) without the need for unpleasant high peak energy



enPuls*Pro*

the next generation

Details

- ports for two hand pieces
- 2 hand pieces
- integrated VAS scale
- intuitive touchscreen operation
- energy levels from 60 to 185 mJ (comparable to 1-5 bar)
- pulse frequencies from 1 to 22 Hz
- burst mode for trigger point treatment
- positive/negative pulse counter with value preset
- SD card for memory, updates and service
- efficient energy transfer
- lower maintenance, longer lifetime
- unique, comfortable pulses (Softshot)

Zimmer
MedizinSystems

... efficient and easy to use

Ergonomic

The high mass applicator reduces strain on clinicians. The ergonomic aluminum hand piece with high mass and ample inertia provides appropriate energy transfer to the patient. It also affords optimal protection against vibration for the healthcare professional. The design allows for single-handed use, enabling the second hand to be free to adjust treatment parameters during the session.



Safe

Pulses are triggered using a foot pedal, ensuring targeted treatment and allowing the clinician to concentrate on hand positioning for the treatment.



Flexible

Applicator heads of varying sizes and shapes can be quickly changed out without the use of tools, allowing treatments to be adjusted as needed.




Convenient

Two simultaneously connected hand pieces increase convenience and save time. The clinician has on-demand access to different applicators heads.

Integrated VAS scale

The integrated Visual Analog Scale allows the therapist to assess the intensity of the patient's pain. This presents both the therapist and the patient with an overview of progress over the course of therapy.

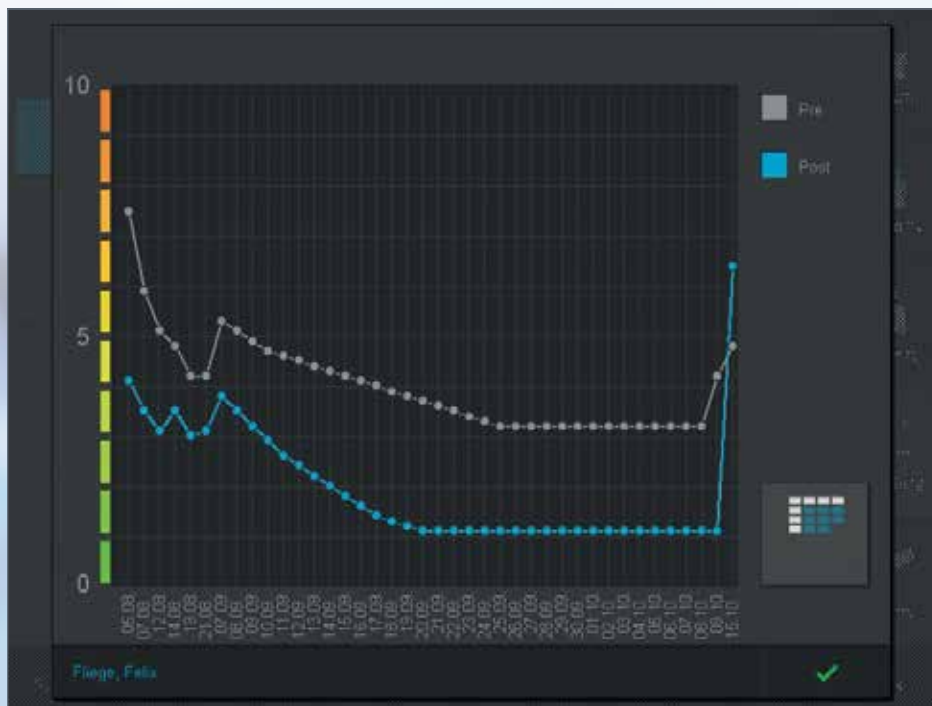

Revolutionary interface design

The new enPulsPro touch screen interface is truly innovative, intelligently designed... and even fun to use.

Larger display surface

The 12" color touchscreen display is easy to read and to navigate with your fingertips.

No air compressor

The enPulsPro is completely portable for use between treatment rooms. There are no messy air compressors to drag, listen to or maintain.

2 million pulse warranty

A powerful, mechanical energy generator with high percussion, digital control, and low mechanical wear provides for an extended lifetime of reliability and performance. We guarantee our hand piece for 2 million pulses.

enPuls**Pro**

the next generation

Successful therapy

Radial Shockwave Therapy:
Easy to operate
Quick results

*The enPulsPro
becomes an
extension of your
hands, delivering
mechanical
stimulation to the
target tissue.*

Radial Shockwave Therapy provides an excellent alternative for treating muscle tissue.

The energy treats muscle pain and help increase the circulation in the target tissue.

Easy-to-use and requires minimal preparation, enabling the practitioner to deliver efficient and effective treatments.

On average, Radial Shockwave Therapy is performed once or twice a week for a total of 5-6 sessions.

Most treatments are administered at 2,500 - 3,000 pulses and take less than 5 minutes per session.



Zimmer
MedizinSystems

Treat soft tissue pain in your office with this cost-effective, non-surgical alternative

It's had a tremendous effect on our practice, enabling patients to get better sooner.

— *D. Lipitz, Physical Therapist*

Pulsed wave therapy is a tremendous asset to my practice. It is the most effective treatment modality that I use to treat patients suffering from chronic foot pain unrelieved by traditional conservative treatments.

— *M. Cunha, Doctor of Podiatric Medicine*

Nothing I have used has been as significant as Radial Pulse Shockwave Therapy. It saves my hands.

— *T. Rodman, Chiropractic Physician*

After two back surgeries and one spinal cord pain stimulator did not relieve my pain, the doctors said I would just have to live with it. Thanks to this treatment, I feel I have recovered at least 80% of my quality of life.

— *J. Pieschacon, 73-Year-Old Patient*



enPuls^{Pro}

the next generation



Zimmer MedizinSystems
3 Goodyear, Suite B
Irvine, CA 92618
(800) 327-3576
info@ZimmerUSA.com
www.ZimmerUSA.com
shop.ZimmerUSA.com

Zimmer
MedizinSystems

enPuls^{Pro} Specifications

Technology	Compressor-free ballistic Radial Shockwave Therapy system with electromagnetic generator for projectile acceleration
Power Levels	60-185 mJ freely selectable in 10 mJ increments
Modes	Frequencies from 1 to 22 Hz 3 Burst modes (4 / 8 / 12 pulses)
Programs	7 preset programs, customizable
Controls	12" color touchscreen for all software operations Rotary control for energy level and for frequency Hand piece operation with multi-directional foot switch
Memory / Updates	SD card for favorites and customized programs (up to 120 entries) Firmware updates
Radial Shockwave Hand Piece	Ergonomic, with anodized aluminum housing and cooling fan
Generator	Electromagnetic, integrated
Dimensions	9.1" L, 2" diameter (max.)
Weight	ca. 1.87 lbs. (with cable)
Warranty	2,000,000 pulses
Applicator heads	Diameters of 6 / 15 (2 pieces) / 25 mm Changeable without tools
Power Consumption	100 - 240 V / 50/60 Hz, max. 250 VA
Dimensions	7.9 x 13.8 x 11.8 inches (L/W/H)
Weight	8.4 lbs. (without hand pieces)
Package	Trolley SysCart including Holder for 2 hand pieces 2 hand pieces Applicator head (6 / 15 (2 pieces) / 25 mm) 10 blue silicone caps Foot switch Mains cable 1 bottle enPuls lotion Tray for applicators

In the interest of product improvement, the specifications and/or appearance of this product are subject to change without notice.

Zimmer MedizinSystems is in no way affiliated with Zimmer Biomet™